---

**SLIM UP**<sup>®</sup>

**BTS**  
beauty total solution

**THAO**<sup>®</sup>

**OPTOS**<sup>®</sup>

**P E R F E C T**<sup>®</sup>  
T O T A L B O D Y

**[ClubWell]**<sup>®</sup>  
relax for active people

**SUNAWAY** <sup>®</sup>

# ... is ULTRA!

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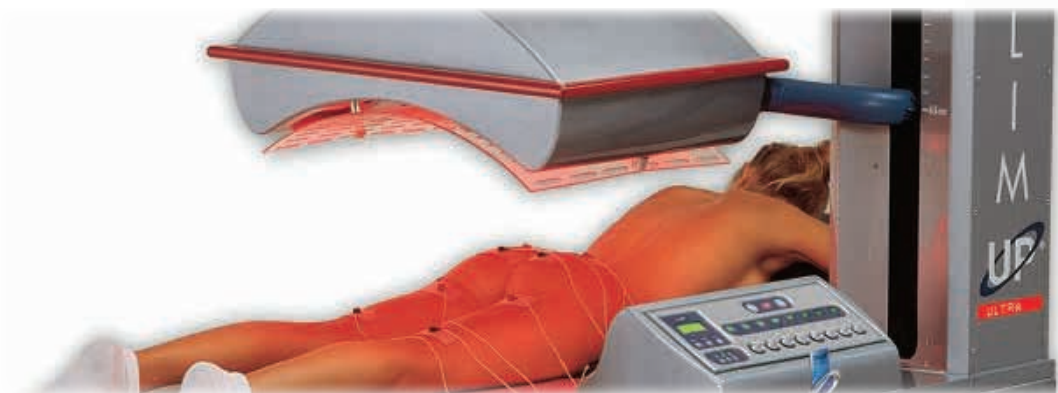
Slim UP® ULTRA offers a complete range of treatments which employ synergetic energies for globally treating unsightly body imperfections. The absolutely unique strength of Slim UP® ULTRA lies in a careful and detailed examination of energy associations and their ability to deal with a cosmetic problem from many directions and with power boosted by these concerted effects. By integrating a widely laboratory tested infrared-electro stimulation treatment with strong ultrasound wave sources represents a decisive innovation in the cosmetic industry, giving the professional beautician an advanced, complex yet easy to use tool: 16 adhesive stimulating electrodes; a full 20 active ultrasound heads distributed over just 4 easy to position plates; 6 infrared ray sources distributed through the use of a height adjustable, mobile arm; 7 resident programs which can be controlled in terms of time and intensity to treat the most common cosmetic problems.

But, Slim UP® ULTRA also represents an important advance when it comes to rejuvenating and firming the face and body through two hand pieces which open up new and fascinating horizons in cosmetic treatment. The face-body hand piece takes advantage of the emission of superficial ultrasound waves at 3 Mhz capable of concentrating the well-known stimulating trophic fibroblast activating effects on the skin layers. However, it particularly favours the diffusion of active substances through the use of specifically formulated gels: tonic and revitalising cosmetic ingredients are used in the anti-age face treatment, but also in the treatment of particular areas of the body, like sagging skin on the inner thigh, arms, belly and breast areas.

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## Function programs

### SLIM Program

A program which integrates infrared, ultrasound and electro stimulation, aimed at slimming in the presence of cellulite\*, in general, and cellulite\* with widespread or localised fat, in particular. The use of electro stimulating frequencies aimed at the fast fibres works without hypertrophying the muscle structure. The ultrasound phase decisively enhances the effect of the infrared rays in the areas richest in fat.

### TONE Program

A program which synergises infrared rays and electro stimulation to obtain improved muscle and skin tone (for example after rapid weight loss or due to a sedentary lifestyle) and for cellulite\* dominated by a loss of skin suppleness. The electro stimulation causes prolonged contractions at frequencies directed mostly on slow and intermediate fibres, resulting in a rapid effect on muscle volumes and on tissue tone.

### MODEL Program

A program based on infrared rays, ultrasound and electro stimulation. The mobilisation of the muscles occurs in two frequencies at medium-low and medium-high frequencies, to more widely act on all types of fibre and to promote both a toning and a slimming effect. Ultrasound, in synergy with infrared rays, actively promotes fat reduction. Therefore it is a program useful in mixed situations, in which both a reducing and a shaping effect are required, as in the case of cellulite\* conditions with loss of tone or presence of fat, or in the case of long term cellulite\* conditions with a surplus of connective fibres and sagging.

### MASS Program

An infrared rays-electro stimulation program which, in the electro stimulation phase prevalently activates the slow and intermediate muscle fibres with maximum, prolonged contractions, thus causing rapid hypertrophy and an increase in targeted volumes. This program is preferred in the treatment of large muscles (loss of tone in the buttocks, relaxed abdominals, shaping and strengthening of the shoulders) when the goal is to globally tone or balance body parts which are not homogeneous.

### LIPOCELL Program

Uses the synergy between infrared rays, ultrasound and electro stimulation. It uses the intensity of light current and very low frequencies which are especially indicated in the capillarization, in improving micro-circulation and drainage. Ultrasound waves are also very delicate with uses that are purely trophic, capillarizing and restructuring. Therefore it is mainly used in the initial phases of cellulite\*, in cellulite\* with initial formation of fat and in lymphedematous cellulite\*. It is also helpful in the case of sluggish circulation.

### FIBROCELL Program

A program which integrates infrared rays, ultrasound and electro stimulation to actively intervene on forms of old, persistent cellulite\* which has slowly accumulated connective fibres which make the tissue fibrous and lacking in resilience. It uses the fibrolytic effect of ultrasound waves and is therefore also indicated in scar-like reactive forms following liposculpture or lecithin treatments.

### MEGASLIM Program

A program which uses ultrasound-infrared synergy, with two complete phases dedicated to ultrasound. It is the most powerful memory resident treatment for fat deposits and is therefore indicated in cases of particularly accentuated adiposity and conditions characterised by well delimited but very fibrous and confined, like trochanteric pads, rolls of fat under the buttocks and "love handles".

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## **Infrared radiations**

- *Beneficial effect on excess of fat and obesity, it stimulates local metabolism thanks to the infrared heating, which penetrates deeper in the body than it does with other methods*
- *Beneficial effect on cellulite\* due to an increasing of circulation, capillarisation, oxygenation and local nutrition, an increasing of detoxification, and vascular gymnastics*
- *Better penetration of cosmetic products due to the increased micro-circulation and to the reduction of the skin electrical barrier*
- *Detoxifying or detoxicant effect due to increased perspiration and the elimination of drugs, toxins, pollutants etc.*
- *Stimulating effect on the immune system and on antiviral defenses*
  - *Analgesic and pain-killing effect*
  - *Muscle relaxing and soothing effect*
  - *Relaxing effect*

## **Steam and aromas**

- *Better absorption of aromas conveyed by steam, through sweat glands which are activated at their most and through sebaceous glands which are freed from sebum*
- *Vessel toning, reducing and firming effect due to the nature of aromatic active principles*

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*Slim UP® STEAM is a beauty therapy unit combining the thermal effect of infrared rays with the beneficial effects of steam and aromatherapy; it features five programmes alternating and combining these functions for obtaining a vessel toning, firming, reducing and slimming effect according to needs; this is an essential unit for the multi-factor treatment of cellulite\*.*

*Slim UP® STEAM helps your customers in 4 ways:*

*A) a draining effect, in case of water retention, oedema, cellulite\* with lymph-oedemas, and excess weight due to the increase of the body liquid component*

*B) a lipolytic effect in case of excess weight*

*C) a beneficial effect in presence of any kind of cellulite\*, due to a better local vascularization, to the improved drainage, to the elimination of toxins and of waste products and to lipolysis*

*D) skin firming effect, due to the deep penetration of essential oils.*

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### 5 – **Lifting Face / 6 – Lifting Neck [Matrix RF]**

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Preset programs

**1 – Biostim [Hyperthermic RF + RED Photobiostimulation + Electrical Stimulation]**

This type of treatment has been studied to have a clear increase of the cellular turnover and therefore the chance to accelerate all those physiological principles that bring to the generation of new epidermal tissue.

**2 – Dermatone [Electrical Stimulation + Hyperthermic RF + Vacuum + Press]**

This treatment protocol has been studied to obtain a strong tonifying effect of derma. The larger electrodes' geometry permits to obtain a wider and embracing treatment of tissues. So, it is specifically applied in all treatments of collagen and all endogenous structural proteins stimulation.

**3 – Miotone [Electromiostimulation]**

Type of treatment studied to stimulate the face muscular layer. Often, together with chronoageing of the skin it is associated a loss of vigour of the underlying muscular layer, with consequent loss of face tone and formation of folds, wrinkles and more evident relaxations. The aim of the treatment is a tonifying effect on the muscles through the stimulation of specific face points with properly studied currents.

**4 – Crono Aging [Vacuum + Hyperthermic RF + IR Photobiostimulation]**

This treatment is suitable for all those situations where the photoageing, through the production of free radicals, causes a thickening, dehydration and ageing of the superficial layer of the skin. The synergy of the technologies allows to have a sum of effects that contribute to the maximisation of the reply and therefore the resolution of the imperfection.

**5 – Lifting Face / 6 – Lifting Neck [Matrix RF]**

This treatment has been studied to solve important problems of face and décolleté skin relaxation.

The special geometry of the electrode and the type of emission manage to evoke a microneedling effect, very effective in deep level on the collagen and Fibroblasts structures, since for its physical specifications it is able to cross easily the epidermis and the surface structures of derma. In fact, thanks to the fractioned signal and the energy emission matrix, it is possible to concentrate the energy at deep levels of derma by thus evoking significant and immediate effects on skin restructuring, by also stimulating an important oxygenating and regenerating action.

**7 – Rebalance [Athermal RF + Electrical Stimulation]**

Program addressed to the rebalancing of the physiological factors of skin. Thanks to the use of two technologies, there is a first phase totally dedicated to the ionic rebalancing of derma and thus granting a normalizing effect and a second phase of nourishment and hydration.

**8 – Epismooth [Hyperthermic RF + Electrical Stimulation]**

The synergy between RF and microcurrents with vehiculating scope allows to obtain a targeted treatment to the mitigation of scar results thanks to the effects of derma shaping and thanks to the effects of the used active ingredients: antioxidants, moisturizing, tonifying. The electrodes used in this treatment can grant a gradual and widespread increase of the thermal energy, able to stimulate effective restructuring replies of derma.

**9 – Acute Acne [Blue Photobiostimulation + Electrical Stimulation]**

Treatment specially targeted to the active phase of acne. With the application of this type of technology it is possible to make both an antibacterial and anti-inflammatory treatment; in fact, by exploiting special light energy emissions some chemical reactions, able to work on the bacterium responsible of acne

(Propium Bacterium Acnes), and then also stimulating skin anti-inflammatory reactions through the interaction of electrical stimulation.

**10 – 3D UP [Hyperthermic RF + Matrix RF + Electrical Stimulation]**

Treatment targeted to fight the most evident face imperfections. The treatment protocol consists of 3 phases, with the use of different stimulations to better exploit the synergy of the effects. Phase 1: medium frequency emission, microcirculation activation and collagen stimulation. Phase 2: fractional radiofrequency emission, better thermal effect and strong deep action. Phase 3: low frequency stimulation, it increases the cell metabolism and allows the use of possible active ingredients in association to increase the antioxidant effect.

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### **Hand-piece 1**

- Head **A** dimensions 35x25 mm

Radiofrequency at emission points Matrix for face treatments

- Head **B** dimensions 52x38 mm

Radiofrequency at emission points Matrix for décolleté treatments

**Matrix RF** (Resistive RF, emission points Matrix)

The special geometry of the electrode and the type of emission manage to evoke a microneedling effect, very effective in deep level on the collagen and Fibroblasts structures, since for its physical specifications it is able to cross easily the epidermis and the surface structures of derma. In fact, thanks to the fractional signal and to the energy emission matrix, it is possible to concentrate the energy at deep levels of derma by thus evoking significant and immediate effects on skin restructuration, by also stimulating an important oxygenating and regenerating action.

- Head **C** dimensions 35x25 mm

IR Led + Hyperthermic RF/Athermal RF/Electrical stimulation

- Head **D** dimensions 35x25 mm

Red Led + Hyperthermic RF/Athermal RF/Electrical stimulation

- Head **E** dimensions 35x25 mm

Blue Led + Hyperthermic RF/Athermal RF/Electrical stimulation

- Head **F** dimensions 35x25 mm

Hyperthermic RF/Athermal RF/Electrical stimulation

**IR Photobiostimulation** (850nm)

Luminous emission able to penetrate deeply on tissues, used to generate a deep diathermic effect able to stimulate the cellular metabolism, by facilitating the cellular turn-over and also the metabolic reduction of the deep fat layer.

**RED Photobiostimulation** (616nm)

Specific luminous emission able to interact with cellular mechanisms by increasing all activities aimed at the improvement of the skin trophism, by promoting the photostimulation of particular mitochondrial proteins.

**BLUE Photobiostimulation** (470nm)

Specific electromagnetic emissions able to have a bactericidal action even against the organisms responsible of acne. In fact, the Blue/Violet light provokes a photosensitization reaction of the porphyrins by producing oxygen and therefore having an anti-bactericidal action.

**Hyperthermic RF** (Bipolar Resistive RF)

Innovative technology able to have anti-aging effects. The generated thermal effects give an immediate tension effect on the skin by acting on the oxygenation and stimulation of the existing collagen, but also long-term stimulation effects of the new structural proteins by the Fibroblasts.

**Athermal RF** (Pulsed Bipolar Resistive)

Thanks to special electromagnetic emissions, this type of RF interacts directly with the skin cellular nature, by obtaining a pH rebalance, and also many tissue humours, by bringing back skin to its normal balance. In this treatment the radiofrequency signal is modulated, so that to have effects directly on specific ionic channels, granting a series of reactions with anti-inflammatory effect and increase of intra/extracellular exchanges.

**Electrical Stimulation** (Electropulse)

Specific electromagnetic emissions able to create some priority channels through the skin, by allowing the penetration on it of specific substances able to have important effects on the whole tissue section. According to the problem, they can be conveyed antioxidants, vitamins, proteins etc.

### **Hand-piece 2**

- Head A dimensions 35x25 mm

- Head B dimensions 52x38 mm

**Vacuum** (max 80kPa)

Derma mechanical stimulation through the use of pulsed vacuum phases, to induce the increase of microcirculation, the drainage and also a series of reactions of subcutaneous tissue re-adjustment, by remodelling the derma fibrousness. The vacuum treatment makes a massage of derma structures by improving the lymphatic circulation and making uniform the tissue consistence.

**Press** (max 150 kPa)

Press is a type of treatment that provides for rhythmic push phases, with air, able to generate some subcutaneous mechanical vibrations. These vibrations, by interacting with the skin receptors, are able to stimulate different effects including toning, microcirculation increase and therefore anti-oxidant effect.

### **Hand-piece 3**

**Electromiostimulation**

Type of treatment studied to stimulate the face muscular layer. Often, together with chrono-ageing of the skin it is associated a loss of vigour of the underlying muscular layer, with consequent loss of face tone and formation of folds, wrinkles and more evident relaxations. The skin drapes on the muscular layer and therefore a loss of tone of this latter will lead to a tired and saggy aspect, also by making less effective the face treatments just finalized to epidermis treatment.

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***The most advanced technologies for face, décolleté and body  
treatments in a single device, in synergy with each other to  
maximize the performance!***

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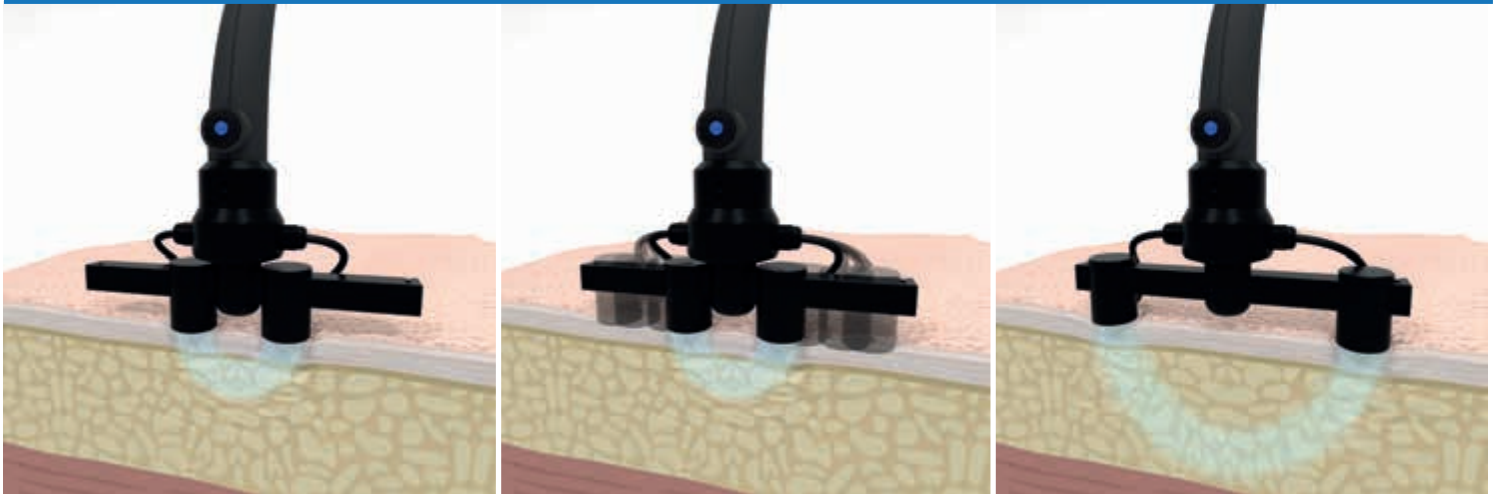
*A part of the treatments is performed with a handpiece characterized by the possibility of mounting different types of heads, with different geometric characteristics and functions*

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*The head with **variable geometry** for body treatments allows, by increasing or reducing the distance between the electrodes, to focus the energy delivered respectively deeper or more superficially to work exactly where you want to concentrate the treatment action.*

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### • Hyperthermic radiofrequency - Bipolar FACE head + bipolar WRINKLES head

*The technique allows an immediate "lifting" effect of the skin thanks to the reorganization of the connective fibers; this rapid effect will be followed by a more lasting one of neofibrillogenesis, with permanent production of new connective tissue. This allows a remodeling of the shapes, a recovery of the skin tone, the reduction of sagging folds, an overall rehydration.*

### • Pulsed radiofrequency - Bipolar resistive

*The radiofrequency emission occurs in a pulsed way, allowing you to work even superficially and in delicate areas or in sensitive parts.*

### • Matrix RF - Resistive hyperthermic radiofrequency with emission points matrix

*The particular "column" emission of energy is exploited to work especially at the epidermal level, concentrating the action on superficial imperfections. Micro-roughness, pigment spots, skin impurities, dilated pores, post-acne scars benefit from the treatment.*

### • Electropulsation

*It helps the absorption of antioxidant, lightening, anti-acne, anti-aging, moisturizing substances, synergizing the action of the various types of radiofrequency chosen.*

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## Body treatments

### • Hyperthermic radiofrequency - Resistive hexapolar

*The hexapolar handpiece allows to broaden and intensify the field of action for the areas of the body. The rise in dermal temperature is exploited in excess skin adiposity and in cellulite\* with an increase in the adipose panniculus. The firming effects remain, thanks to the collagen and elastic neofibrillogenesis, for which application on sagging and hypotonic areas and on cellulite\* with loss of tone is indicated. The improvement of microcirculation and the oxygenating,*

*detoxifying and draining action have excellent indications in cellulite\* with stasis, edema and water retention.*

### • Pulsed radiofrequency - Resistive hexapolar

*In this case, the hexapolar handpiece works more superficially and is preferred for the treatment of the so-called cellulite\* in the early stages or in the presence of water retention and orange peel. Stretch marks with skin that has lost firmness are also an elective indication.*

### • Matrix RF - Hyperthermal resistive emission dot matrix radio frequency

*The particular "column" emission of energy is used to work especially at the epidermal level, concentrating the action on superficial blemishes. The blemishes of stretch marks, pigmentation spots and skin impurities find fundamental benefit from the treatment.*

### • Electropulsation

*Especially in the body, where the thickness of the skin and the depth of the fat make aesthetic interventions difficult, it is important to combine the effect of targeted active ingredients that synergize the action of other physical energies. raining, firming, lipolytic, anti-cellulite\* cosmetics, etc. are indicated.*

**Easy Power**  
Body & Face

## **DEMS – Diffuse Electro Myo Stimulation**

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## **DEMS – Diffuse Electro Myo Stimulation**

*Method that uses electrical impulses of various frequency, intensity and duration to obtain a targeted contraction on certain muscle bundles, but also widespread on all minor, voluntary and involuntary muscle bundles present in the treated area. To obtain a correct and pleasant muscle movement for the treated subject, the waveform is biphasic compensated.*

**DEMS is applied to the muscles with an innovative system of electrically conductive bands which allow an ideal diffusion of stimulation in the treated area and allow the treatment to be concentrated on any areas with greater need.** *It spread the stimulation over multiple muscle bundles and make the physical activity generated more physiological. By varying the frequency, last of the pulse and the pause, the treatment is aimed more at aerobic and therefore slimming rather than anaerobic and therefore toning rather than microcirculatory activities.*

*This technique allows you to perform a contraction without tiring (therefore for all ages) and make work both voluntary and involuntary fibers.*

*A DEMS phase carried out after a HEM Radio wave phase benefits from the endogenous heating generated in the area and allows to use high intensity electrical stimulation without discomfort.*

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### *DES – Draining Electro Stimulation*

*Technology essentially aimed at **venous and lymphatic drainage, extremely suitable for blemishes due to excessive stagnation of localized liquids**. It takes advantage of the electrical impulses to perform a peristaltic massage through the sequential activation of the stimulating channels, which create a significant increase in lymphatic migration towards the lymphatic drains.*

*Essential treatment also to drain the body after phases aimed at slimming, toning and multifactorial treatment of cellulite\*.*

***It is applied with an innovative system of soaked electroconductive bands.***

*The treatment consists of two phases:*

- *Phase A: characterized by electromagnetic stimulation capable of increasing peripheral circulation capillarization phase.*
- *Phase B: characterized by a peristaltic massage performed through the sequential activation of the stimulating channels.*

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*Hi-tech equipment based on latest generation technologies to treat the imperfections of: face, décolleté and body. Equipped with three face and body handpieces that can act in synergy by delivering different active energies and an electromyostimulation section for the body.*

*According to the area to be treated, modalities, duration, intensity and type of intervention are selected, to work in a targeted way and obtain a synergistic increase in the effects.*

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#### **• Hyperthermic radiofrequency - Bipolar resistant**

The technique allows an immediate "lifting" effect of the skin thanks to the reorganization of the connective fibers; this rapid effect will be followed by a more lasting one of neofibrillogenesis, with permanent production of new connective tissue. This allows a remodeling of the shapes, a recovery of the skin tone, the reduction of sagging folds, an overall rehydration.

#### **• Pulsed radiofrequency - Bipolar resistive**

The radiofrequency emission occurs in a pulsed way, allowing you to work even superficially and in delicate areas or in sensitive parts.

#### **• Matrix RF - Resistive hyperthermic radiofrequency with emission points matrix**

The particular "column" emission of energy is exploited to work especially at the epidermal level, concentrating the action on superficial imperfections. Micro-roughness, pigment spots, skin impurities, dilated pores, post-acne scars benefit from the treatment.

#### **• Electropulsation**

It helps the absorption of antioxidant, lightening, anti-acne, anti-aging, moisturizing substances, synergizing the action of the various types of radiofrequency chosen.

### **Body treatments**

#### **• Hyperthermic radiofrequency - Resistive hexapolar**

The hexapolar handpiece allows to broaden and intensify the field of action for the areas of the body. The rise in dermal temperature is exploited in excess skin adiposity and in cellulite\* with an increase in the adipose panniculus. The firming effects remain, thanks to the collagen and elastic neofibrillogenesis, for which application on sagging and hypotonic areas and on cellulite\* with loss of tone is indicated. The improvement of microcirculation and the oxygenating, detoxifying and draining action have excellent indications in cellulite\* with stasis, edema and water retention.

#### **• Pulsed radiofrequency - Resistive hexapolar**

In this case, the hexapolar handpiece works more superficially and is preferred for the treatment of the so-called cellulite\* in the early stages or in the presence of water retention and orange peel. Stretch marks with skin that has lost firmness are also an elective indication.

#### **• Electropulsation**

Especially in the body, where the thickness of the skin and the depth of the fat make aesthetic interventions difficult, it is important to combine the effect of targeted active ingredients that synergize the action of other physical energies.

Draining, firming, lipolytic, anti-cellulite\* cosmetics, etc. are indicated.

#### **• Electromyostimulation**

It is equipped with 8 pair of electrodes with adjustable intensity, in such a way as to allow optimal coverage of the most important muscle groups or to localize, with the appropriate electrodes, the work on specific areas of the body.

The resident programs carry out various programmable activities: aerobic work to slim down, anaerobic work to strengthen and firm up, stimulation of neovascularization to oxygenate, decontracting and relaxing effect.

In their development, they provide a first preparatory phase of RF of heating and relaxation, followed by real electromyostimulation that mimics the correct physiology of aesthetic training: warm-up-work-cool-down.

This section is an independent operator and can work independently for all situations of loss of muscle and skin tone, in firming, in the so-called cellulite\* with hollows and sagging, in the hypertrophic modeling of well-defined areas but it is ideal in association with radiofrequency and electropulsation handpieces for its boost and synergistic effect.

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**PERFECTA TOTAL BODY** is a unit for total body drainage and presso-massage: it is designed for three different sections, the head and face, the upper limbs and the lower limbs.

*A sophisticated yet user friendly electronic unit, controlled by a microprocessor, manages all the PERFECTA TOTAL BODY functions.*

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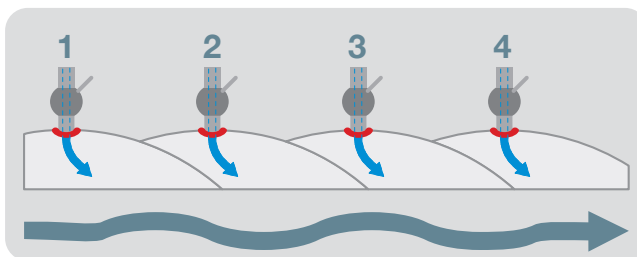
**Sea Wave** is a system made up of overlapping segments: each of these is sequentially inflated before the previous one is totally deflated. This enables a selective and progressive pressure without interruptions.

## THE 3 WORKING SECTION OF PERFECTA TOTAL BODY, usable individually

### A - Head-face section

Helmet with a built-in headrest for sequential treatment of the face (chin, cheeks, forehead, front and back of neck. Front and back of neck with *Sea Wave* System).

The headrest has 6 outlets for a correspondent number of sectors in the helmet, as well as a system for immediate connection to the machine (RAPID-AIR).

### B - Upper limbs section

2 arm-pieces with 7 *Sea Wave* sectors for the sequential treatment of upper limbs and an integrated palm-piece for the hand in order to exercise pressure solely on the palm and on the back of hand with no crushing at the sides.

The particular anatomy of the arm-piece, which is adjustable through a velcro fastening, additionally to the palm piece, allows you to treat the limb uninterruptedly from the hand to the upper shoulder. This section connects to the machine with a quick coupling system, capable of recognising the corset element which it is dedicated to (RAPID AIR).

### C - Section for lower limbs, hips, groin

2 leg-pieces, 2 thigh-pieces, 2 groin compressors, 1 corset element for treating hips and buttocks. Each leg-piece consists of 4 *Sea Wave* sectors and of an arch support made by a sole of self-moulding material, a rigid shaped sole and an air bladder, designed for exercising pressure under the sole of the foot and never at its sides.

The pressure is applied from the sole of the foot all along the limb with no interruption.

The thigh-piece makes it possible to adjust the extension of the leg-piece and to adapt it to the different heights of the limbs. The anatomically designed groin compressor allows a perfect lymphatic drainage of the groin area.

The corset for hips and buttocks is made of one piece only and consists of 2 *Sea Wave* sectors.

The adjustment of the corset elements through a velcro fastening allows you to extend them to very different sized anatomical parts.

This section connects to the machine with a quick coupling system, capable of recognising the corset element which it is dedicated to (RAPID AIR).

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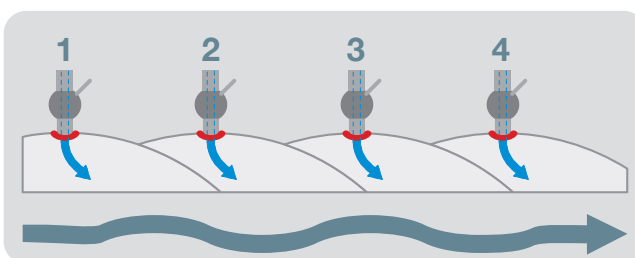
***PERFECTA TOTAL BODY** is a unit for lower limbs drainage and presso-massage.*

*A sophisticated yet user friendly electronic unit, controlled by a microprocessor, manages all the **PERFECTA SEA WAVE** functions. 10 preset programs; adjustable in every parameter.*

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### **Lower limbs, hips, groin.**

*2 leg-pieces, 2 thigh-pieces, 2 groin compressors, 1 corset element for treating hips and buttocks. Each leg-piece consists of 4 Sea Wave sectors and of an arch support made by a sole of self-moulding material, a rigid shaped sole and an air bladder, designed for exercising pressure under the sole of the foot and never at its sides.*

*The pressure is applied from the sole of the foot all along the limb with no interruption. The thigh-piece makes it possible to adjust the extension of the leg-piece and to adapt it to the different heights of the limbs. The anatomically designed groin compressor allows a perfect lymphatic drainage of the groin area.*

*The corset for hips and buttocks is made of one piece only and consists of 2 Sea Wave sectors. The adjustment of the corset elements through a velcro fastening allows you to extend them to very different sized anatomical parts.*

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### ***Functional furnishings for beauty and well-being.***

*The ability to create an environment is the essential task of every component.*

*Whether furnishing a beauty treatment booth or a relaxation area or well-being centre. THAO offers numerous targeted solutions with high technological content and great attention to value and craftsmanship as regards the materials used. The original and dedicated design is closely integrated with the functions of the various elements and with the environment in which these are located.*

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*The concept of well-being is becoming increasingly more important for each and every one of us.*

*Every beauty and well-being centre is showing its commitment by offering better services in terms of skill and professional capacities, adopting top-quality and comfortable instruments, the value of which is immediately perceived by clients.*

*THAO MASSAGE comprises the functional furnishings of a beauty treatment booth: couch, trolley, stool, wall-mounted chromotherapy lights, wall mirror.*

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Piano inferiore d'appoggio. LxPxA cm 207x66x80

**THAO MAS.** *This is the most technical couch of the MASSAGE line.*

*Thanks to its original structure, the angle of all the padded surfaces can be changed at will with just a few simple movements, to adapt to all possible treatments. Face hole.*

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230 Vac – A. 2,1 max. Gestione automatica computerizzata. LxPxA cm 144x16x65

**THAO C.** *Wall-mounted chromotherapy lights. Finely-crafted prized wood structure.*

*Minimalist design to better adapt to any environment. The alternation of the lights, the evanescence, the explosion of colours contribute to well being and to the joie-de-vivre of those involved. 230 V AC – A. 2.1 max. Automatic computerised control WxDxH cm 144x16x65*

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Ruote pivotanti con freno. LxPxA cm 84x45x92

**THAO KA.** *Trolley in glass, steel and finely crafted prized wood. Castors with brake.*

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LxPxA cm 100x3,5x60

**THAO I.** *Shaped wall mirror unit. Extends space. Completes and enhances the environment.*

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LxPxA cm 40x40x49

**THAO SI.** *Practical and sturdy stool in steel and finely crafted prized wood.*

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*Sophisticated styling, convenience, choice quality materials, finely crafted exquisite woods resistant to moisture and heat; tough padding and prestige upholstery. Motor-operated infrared device, positioned as required for in-depth heating of the treated zones. 6 IR lamps with independent on-off control, 20'/60' timer, temperature sensor, red safety button.*

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*Brightly coloured, soft, comfortable, washable mattress; see-through, washable, single-client cover.*

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*Input voltage: 100-240 V~*  
*Main frequency: 50/60 Hz*  
*Power consumption: 100 W*  
*Electrical safety Class I*  
*Applied accessories type BF*  
*Continuous operating device*

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*Input voltage: 230 V~*  
*Main frequency: 50/60 Hz*  
*Power consumption: 400 W*  
*Electrical safety Class I*  
*Applied accessories type BF*  
*Continuous operating device*

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*Input voltage: 230 V~*  
*Main frequency: 50/60 Hz*  
*Power consumption: 2,1 A Max*

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*Input voltage: 230 V~*  
*Main frequency: 50/60 Hz*  
*Power consumption: 700 Watt*

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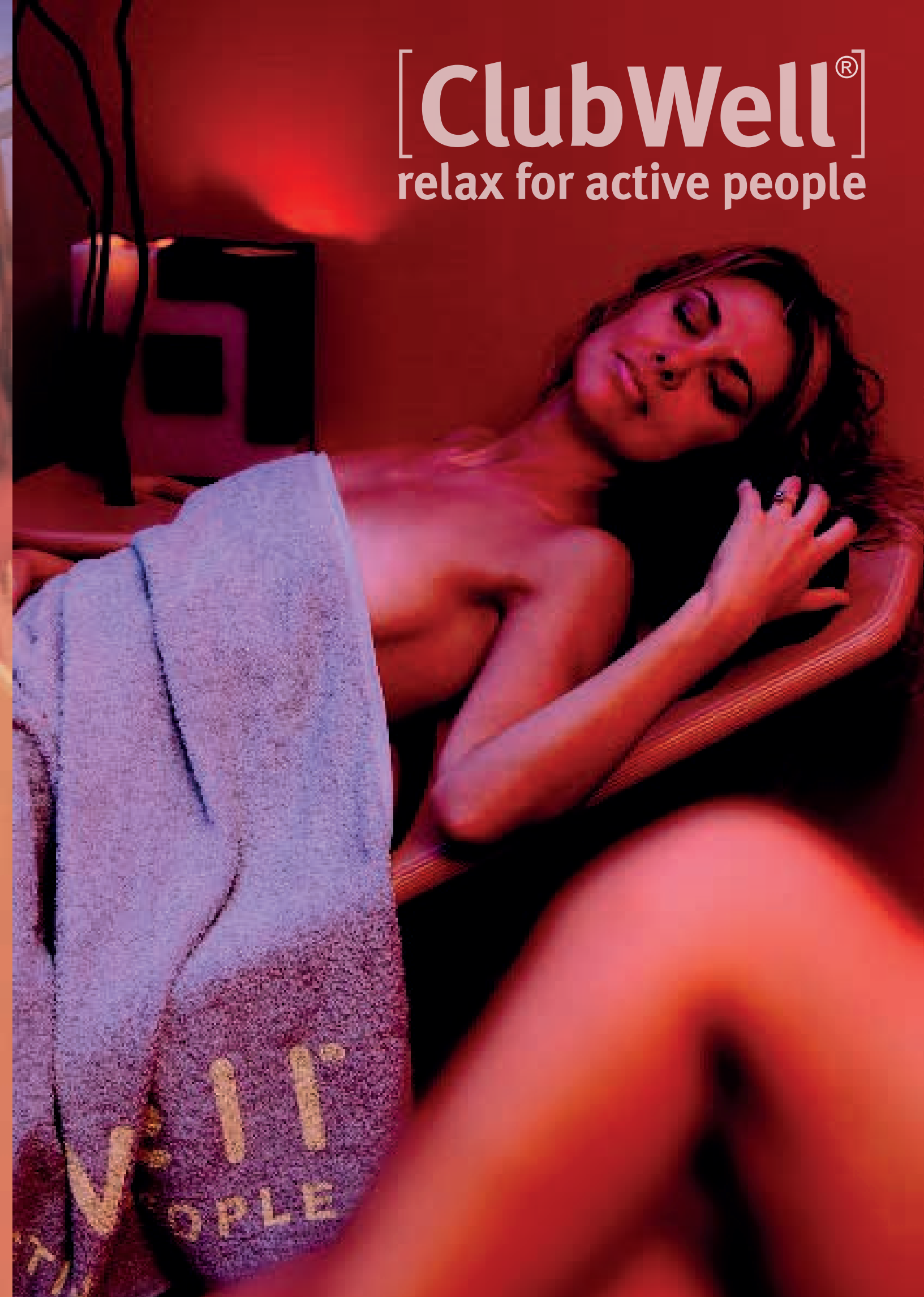
*This company has a Quality System certified according to UNI EN ISO 9001:2015 - UNI CEI EN ISO 13485:2016 standards by the TÜV Rheinland Italia S.r.l. accreditation body*

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**DAM S.r.l.**
DAM S.r.l. E' Azienda certificata dall'ente accreditato TÜV Rheinland Italia S.r.l.